




Stille artery forceps

Stille artery forceps

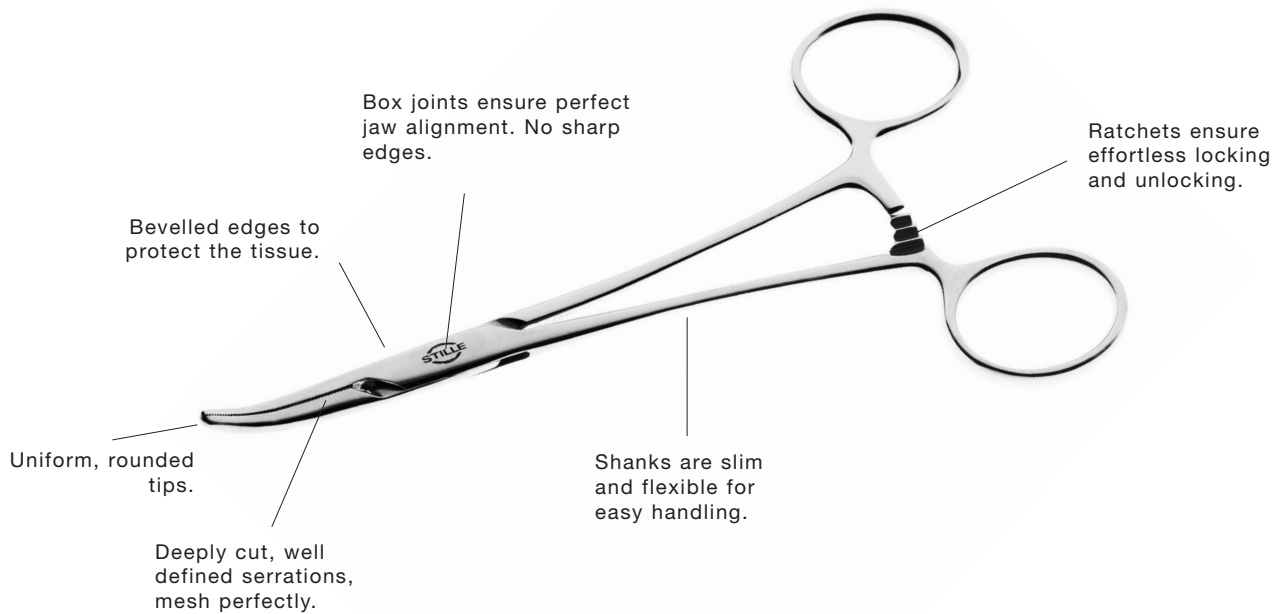

Stille artery forceps are manufactured in stainless steel of the highest quality available and are designed with the specific needs of the surgeon in focus. They are always easily opened and closed, enabling perfect control. The steady grip ensures that the haemostatic forceps does not slip off during surgery. Serrations mesh perfectly. When a bleeding vessel is clamped, blood flow is arrested and no valuable time is lost in making adjustments. The long life –surpassing that of other instrument brands- means reduced costs for the hospital.

How to test Stille Artery Forceps:

Close the jaws slowly until they meet.
If they overlap, the jaws are out of alignment.

If the instrument has teeth, inspect to ensure that the teeth are meshing properly.

While holding the rings of the instrument in each hand, open the instrument as far as possible. If the box lock has considerable play and it is very loose, this will cause misalignment. If this occurs, the instrument is faulty and should be repaired.



Cleaning and sterilizing of a haemostat should always be carried out with the ratchet open.



HALSTED-MOSQUITO DELICATE

Delicate hemostat without teeth for small operating field. Ideal for hand and facial surgery.


- * 103-10026-121 Straight 12 cm 4.75"
- 103-10028-121 Curved 12 cm 4.75"

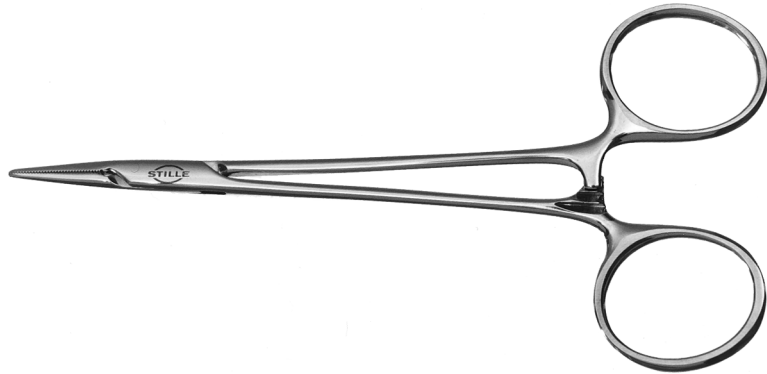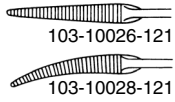

HALSTED-MOSQUITO

Hemostat without teeth, used in small operative fields. For plastic surgery, hand or any delicate surgery.


- * 103-10030-121 Straight 12 cm 4.75"
- 103-10032-121 Curved 12 cm 4.75"
- 103-10032-181 Curved 18 cm 7"

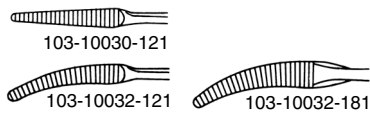

HALSTED-MOSQUITO

Hemostat with 1x2 teeth, used in small operative fields. For plastic surgery, hand or any delicate surgery.

- * 103-10035-121 Straight 1x2 Teeth 12 cm 4.75"
- 103-10037-121 Curved 1x2 Teeth 12 cm 4.75"

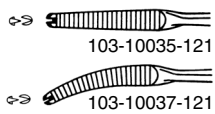

ARTERY FORCEPS


* Photo shows this instrument. Not shown at actual size



PÉAN

Hemostat with long jaws and relatively blunt tip. Widely used when a short instrument is needed.


* 103-10110-141 Straight 14 cm 5.5"

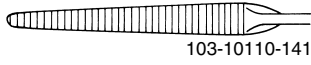

KOCHER

Hemostat with 1x2 teeth. Widely used when a short instrument is needed.





* 103-10115-141 Straight 1x2 Teeth 14 cm 5.5"

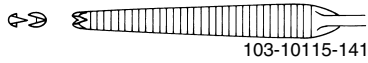

CRILE

General purpose hemostat without teeth. Used in relatively small operating fields, when more strength is needed than a Mosquito can offer. Popular in facial surgery.

- * 103-10180-141 Straight 14 cm 5.5"
- 103-10180-161 Straight 16 cm 6.25"
- 103-10180-181 Straight 18 cm 7"
- 103-10182-141 Curved 14 cm 5.5"
- 103-10182-161 Curved 16 cm 6.25"
- 103-10182-181 Curved 18 cm 7"

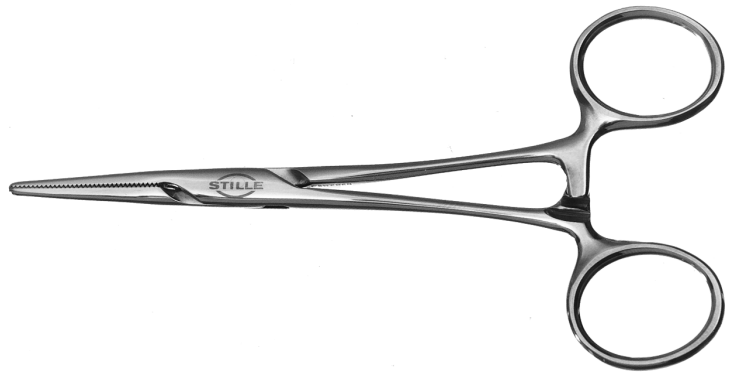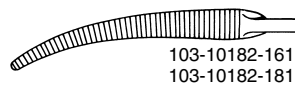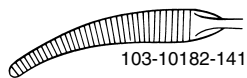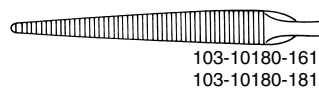

* Photo shows this instrument. Not shown at actual size.


ARTERY FORCEPS



CRILE

General purpose hemostat with 1x2 teeth. Used in relatively small operating fields, when more strength is needed than a Mosquito can offer.




- * 103-10185- Straight 1x2 Teeth 14 cm 5.5"

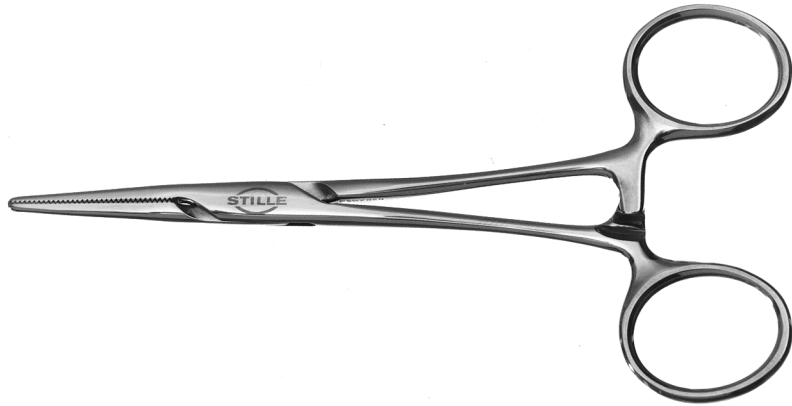

LERICHE

Fairly tapered hemostat. For use in small operative fields.




- * 103-10190- Straight 15 cm 6"
- 103-10192- Curved 15 cm 6"
- 103-10195- Straight 1x2 Teeth 15 cm 6"
- 103-10197- Curved 1x2 Teeth 15 cm 6"

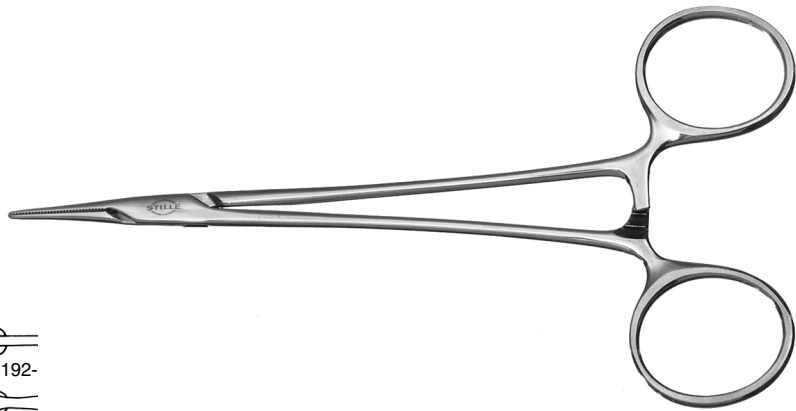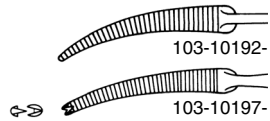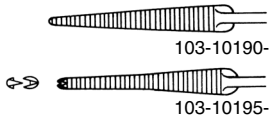

ROCHESTER-PÉAN

Sturdy hemostat. Relatively blunt tips. Used in all types of surgery.

- * 103-10200-161 Straight 16 cm 6.25"
- 103-10200-181 Straight 18 cm 7"
- 103-10202-161 Curved 16 cm 6.25"
- 103-10202-181 Curved 18 cm 7"

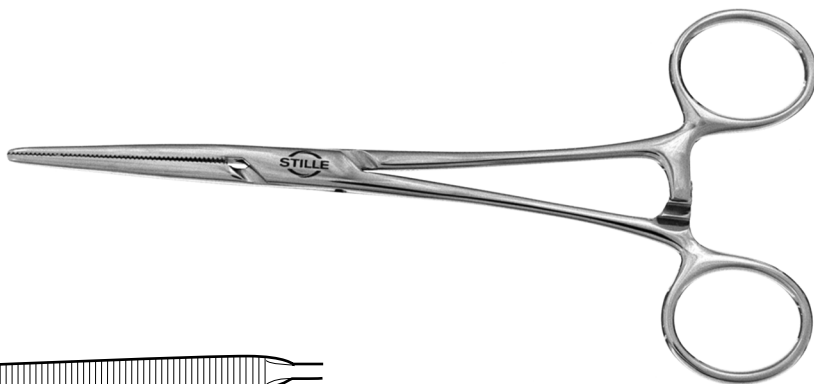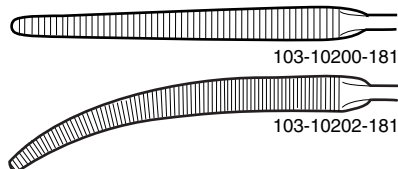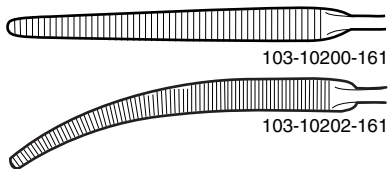

ARTERY FORCEPS

* Photo shows this instrument. Not shown at actual size




ARTERY FORCEPS

ROCHESTER-PÉAN

Sturdy hemostat. Relatively blunt tips. Used in all types of surgery.


- * 103-10210-201 Straight 20 cm 8"
- 103-10212-201 Curved 20 cm 8"

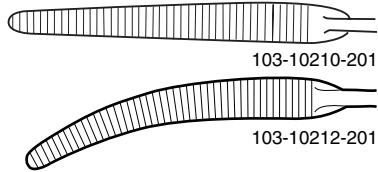

ROCHESTER-OCHSNER

Sturdy hemostat with 1x2 teeth. Used when a strong forceps is needed.


- 103-10205-161 Straight 1x2 Teeth 16 cm 6.25"
- * 103-10205-181 Straight 1x2 Teeth 18 cm 7"
- 103-10207-161 Curved 1x2 Teeth 16 cm 6.25"
- 103-10207-181 Curved 1x2 Teeth 18 cm 7"

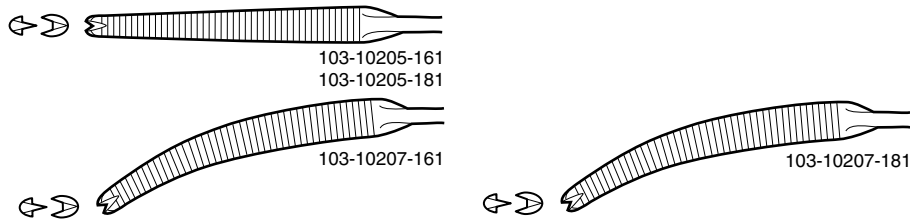

ROCHESTER-OCHSNER

Sturdy hemostat with 1x2 teeth. Used when a strong and long forceps is needed.

- * 103-10215-201 Straight 1x2 Teeth 20 cm 8"
- 103-10217-201 Curved 1x2 Teeth 20 cm 8"

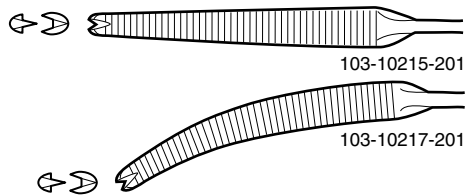




* Photo shows this instrument. Not shown at actual size.



STILLE DISSECTING FORCEPS

Four models of delicate dissecting forceps with different jaw curvature. Useful in neck and breast surgery.



- * 103-10423-1 Slightly curved 20 cm 8"
- 103-10423-2 Curved 20 cm 8"
- 103-10423-3 Strongly curved 19.5 cm 7.75"
- 103-10423-4 Angled 90 deg. 19 cm 7.5"

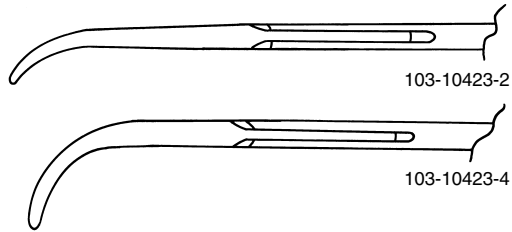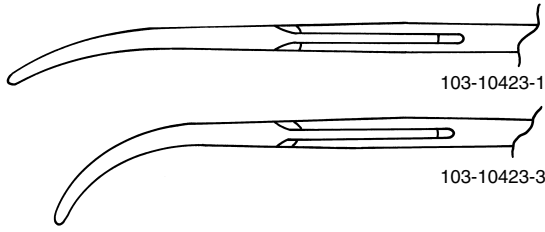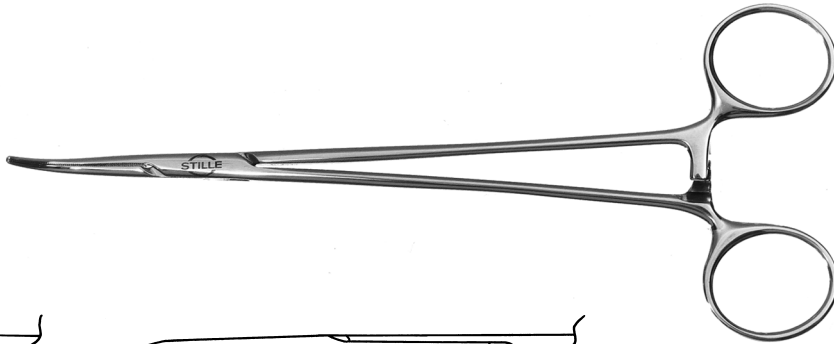

OVERHOLT

Strongly curved dissecting forceps. Jaws half-way serrated.

- * 103-10425- Strongly curved 14 cm 5.5"

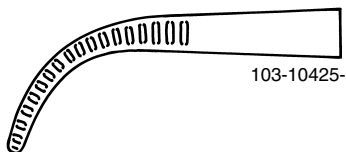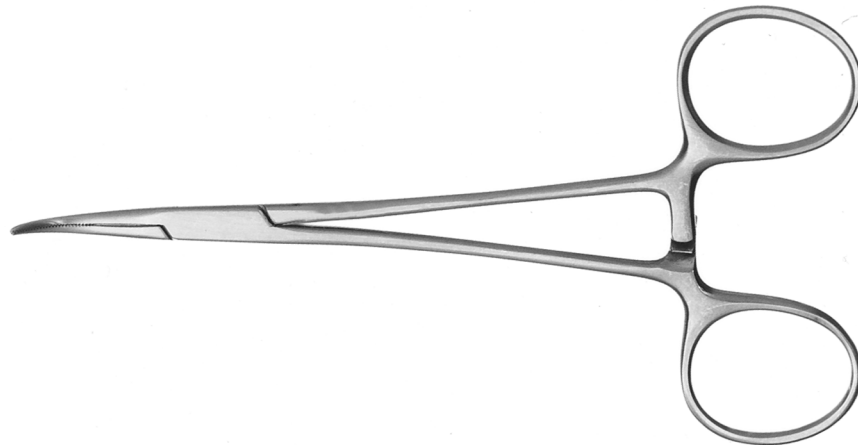

ARTERY FORCEPS

* Photo shows this instrument. Not shown at actual size